




Android Phone Apps

Navigate to you google play store and search: iCare HOME and iCare PATIENT.

Download and install both of these apps.

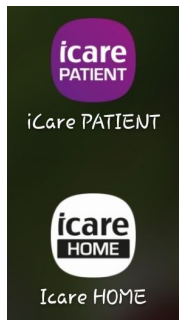

The PATIENT app after you install, can be hidden from your home screen. You never even need to open it or bother with it at all. It will stay asleep until you plug in your Device. Once you do that, the app will automatically open, and upload measurements instantly (usually 2-3 seconds).

The HOME you can use as a quick start and troubleshooting guide right on your phone. This will help you take readings properly, and when used together with our website you will have all the information that you need to complete your own measurements with ease.

NOTE

When you are connecting your device to the phone, you have to use the included transfer data cable. There are two ends both of which are marked.

1. The end marked HOST must be plugged into the phone.
2. The end marked CABLE CONNECTION must be plugged into the device.